




# INSTALLATION OF STI-7532, STI-7533 & STI-7533AED PROTECTIVE CABINET

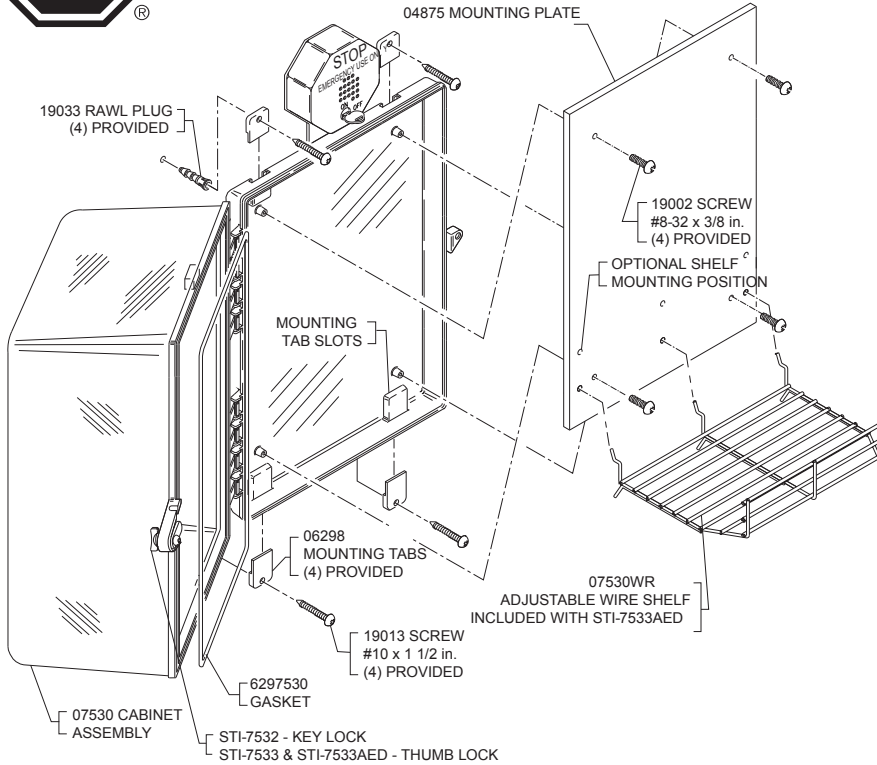

## INSTALLATION OF CABINET

1. Insert the (4) 06298 mounting tabs fully into back box mounting tab slots. Use rubber mallet if necessary.
2. After selecting a suitable location, place cabinet in closed position, against the wall. Mark and drill (4) 1/4 in. diameter holes. Insert the (4) 19033 rawl plugs.
3. Select one of the two shelf mounting locations, install shelf. Close and lock cabinet.
4. Place cabinet in position making sure the (2) switch wires are not pinched between the wall and cabinet. Drive in the (4) screws. Cabinet is now secure.

## INSTALLATION OF ALARM ASSEMBLY

1. Place the alarm mounting plate on the top center of cabinet. Select at least two of the four optional mounting holes. Mark and drill 3/16 in. holes and install the 19018 anchors.
2. Fish the magnetic switch wires through the alarm mounting plate and the siren backplate (View A). Next, drive the two alarm mounting screws into the previously installed rawl plugs.
3. Install the two magnetic switch wires following the terminal switch diagram (View B).
4. Install the alarm cover using the (2) tamperproof screws and provided tool.

